

la belle

la belle<sup>2</sup>bruckhoff  
hannover

## We warmly welcome you to the world of digital hearing

*With this hearing instrument you have acquired a top quality product proudly "Made in Germany".*

*These instructions will quickly help you to become familiar with it.*

*We wish you much hearing pleasure.*

bruckhoff  
hannover

### General tips for use and care

#### ■ Tips for care for your hearing instrument

Your hearing aid specialist provides various cleaning materials. Please take care that no liquids get into the hearing instrument.

#### ■ A few tips for daily use

- Protect the hearing instrument from heat and moisture.
- Avoid dropping your hearing instrument.
- Remove the battery when not using the hearing instrument for a longer period of time.

#### ■ Remove the hearing instrument when doing any of the following

- washing or showering
- using a hair drier hood or hair drier
- using hair spray
- receiving high frequency or ultrasonic treatment

#### ■ Disposal

Do not dispose old instruments and batteries with your household waste. Return them to your hearing aid specialist. He will be glad to do it for you properly.



#### ■ Service and guarantee

If your hearing instrument does not work, please consult your hearing aid specialist after having checked the following:

- Is the hearing instrument switched on?
- Does your instrument work with a new battery?

**Do not carry out repairs yourself.**

### ⚠ General warnings

- Keep batteries away from children and pets. They can be harmful if swallowed. If happened, seek medical help immediately.
- Do not use your hearing instrument in places with a risk of explosion.
- Only use a hearing instrument prescribed personally for you. Other instruments might be ineffective or damage your hearing.
- X-rays can damage your hearing instrument.
- Rearward warning signals may possibly be heard inadequately in the case of hearing aids with multi-microphone technology.

AUDIA AKUSTIK GMBH  
Herrenstraße 6  
30159 Hannover  
Germany

Tel: +49 (0) 511 - 167 00 - 0  
Fax: +49 (0) 511 - 167 00 - 50  
service@bruckhoff.com

[www.bruckhoff.com](http://www.bruckhoff.com)



la belle

la belle<sup>2</sup>bruckhoff  
hannover

## Instructions

### The three main elements of the la belle and la belle 2 hearing aid glasses

1 The glasses with earpieces 2 Click-System 3 Hearing module with 3.1 control unit.  
The hearing module can be easily plugged in or removed using the **click-system**.

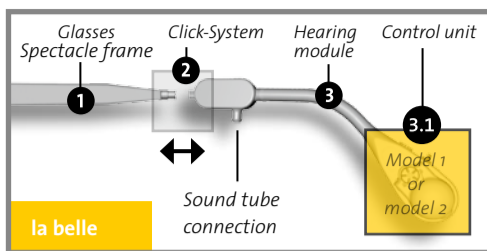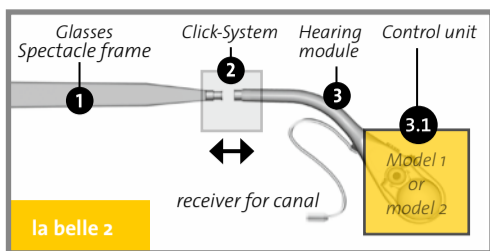


### Functions of the control unit for ...

3.1


### ... la belle and la belle 2

#### Hearing program button (Model 1)

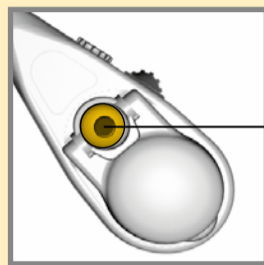

Hearing program button


#### Hearing program button (Model 2)

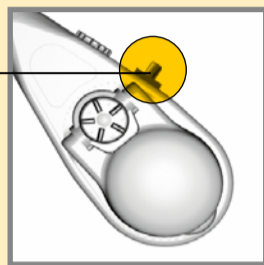

Hearing program button


Pushing the **hearing program button**, you can set the hearing programs pre-installed by your hearing fitting specialist.

#### Volume control (Model 1)

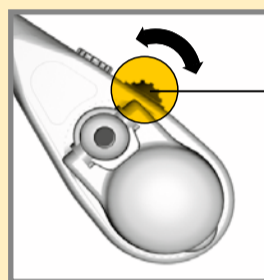

Volume control

#### Volume control (Model 2)

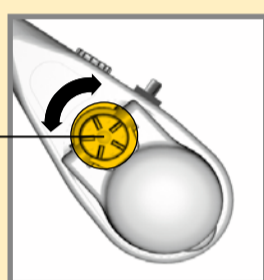


Volume control



You can adjust the volume of your hearing instrument to your personal needs with the volume control.

*Note: Not all models feature a volume control or hearing program button.*

### Switching the hearing instrument on and off (all models)

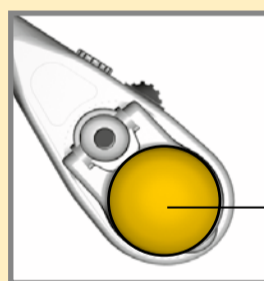

On / off switch  
Battery flap

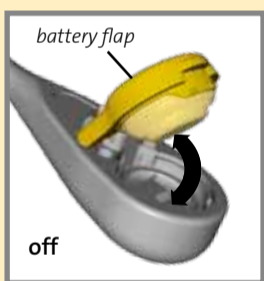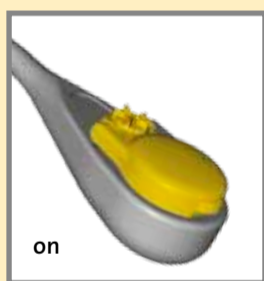


**The same in all hearing modules:** Switching the hearing instrument on and off is extremely simple. When the battery flap is closed the instrument is automatically switched on. You switch it off simply by opening the battery flap.

### Removing the battery (all models)




#### Instructions:

Open the battery flap. Remove the battery from the battery flap using the magnetic rod.

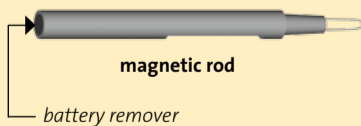

### Inserting the battery (all models)

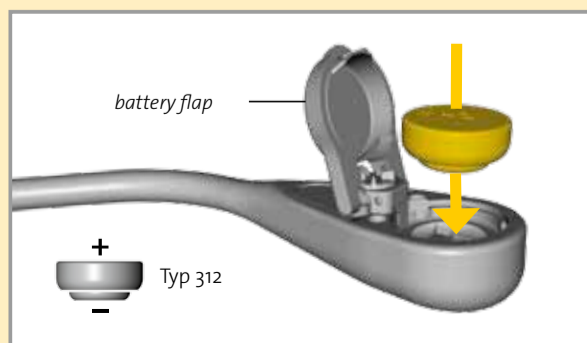

#### Instructions:

Insert the new battery (type 312) into the battery compartment or into the magnetic battery flap with the **“+”-symbol face up** and close the flap – the hearing instrument is automatically switched on!

Attention: In case of CROS/BICROS systems two batteries type 312 are needed.